



Instructions

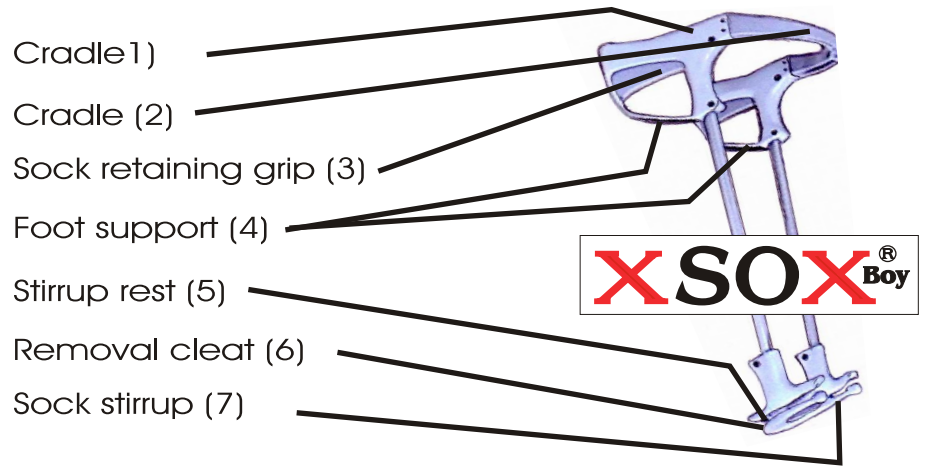

Place cradle (1 + 2) of XSOX boy on the ground with the bow facing forward. Place one or both feet on the foot supports (4) provided or on the forward bow.

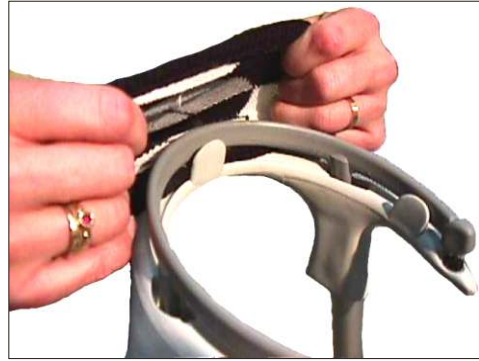

Expand sock opening and place under side of opening on the stirrup rest (5) and



proceed to extend the upper side of sock over the sock stirrup (7).
The heel of the sock should now be facing you.

Important!

To insure troublefree use of the XSOX boy it is recommended that socks or stockings with wide or reinforced necks should be used.





Now turn the XSOX boy upside with the cradle bow facing outwards.



Pull both sock retaining grips (3) upwards so that the socks can be held firmly in place.



Direct the XSOX boy downwards toward the ground and place foot in sock or stocking and pull upwards until sock is fully in place. Then release sock retaining grip (3).

Undressing

Place the XSOX boy stirrup rest at the rear of the leg at the top of the sock and guide the removable cleat (6) between leg and sock.



Pull both sock retaining grips (3) upwards and push the XSOX boy down until the sock or stocking is entirely removed.



Release retaining grips, turn the XSOX boy upside down and remove sock from cleat.

